# **XO ODONTOSURGE**

# EXTRAORDINARY DENTISTRY

# XO ODONTOSURGE

At XO CARE we empower dentists to perform the very best treatments on calm patients, while protecting the dentists' health and finances.

## Your everyday tool for soft tissue handling

Cutting or excavating soft tissue is a daily procedure for most dental professionals. One of the most time-consuming elements of those treatments is managing bleeding and maintaining clear vision.

XO ODONTOSURGE is your everyday tool for soft tissue handling. It is known for perfect healing and faster treatments. The device is always ready to use and makes perfect, dry incisions.

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# PROFESSIONAL DENTISTRY

XO empowers you to perform the very best dental treatments, enabling you to make full use of your skills and expertise.

To remain focused on the work area, you need a cutting device that makes incisions with little or no bleeding.

## Precise incisions

With XO ODONTOSURGE the quality of your incisions is always perfect. The power is automatically adjusted to the resistance of the tissue. The electrode glides smoothly during a cut and you do not experience burned tissue or sticking electrodes.

## Dry conditions for faster treatments

Dry incisions obtained with XO ODONTOSURGE create ideal conditions for taking impressions and composite restoration. You also reduce duration of your treatments and the healing time.

## Individual

According to your preferences, you choose between various straight, loop and coagulation electrodes. Our tips guarantee perfect targeting of the treatment area and allow access to all surfaces.

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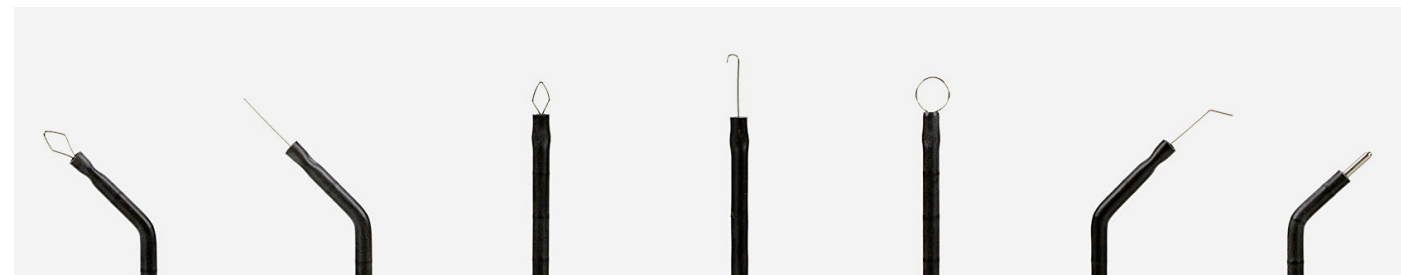
DDS Esben Kardel  
Oslo, Norway

# CARING DENTISTRY

We understand and recognize how important it is that your patient remains calm throughout treatment. To this end, XO's equipment is designed to be safe and comfortable and to help you gain the complete confidence of your patient.

## Safe

To maximize your patient's safety XO ODONTO-SURGE automatically reduces power output when you touch hard tissue. Your incisions will never cause tooth or bone necrosis.

## Superb patient comfort

The perfect and dry incisions create a comfortable experience for your patients. The healing is much faster and there is little scar tissue.

Patient's anxiety is further reduced as XO ODONTO-SURGE works without a neutral electrode.

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# ERGONOMIC DENTISTRY

Your health is one of our main priorities when we design our solutions. We aim to ensure that you sit well, see well and are able to work well – all at the same time.

To create the best possible ergonomic situation, we have integrated the activation button into the handpiece to avoid additional foot or knob controllers.

## Maximum tactility

To minimize strains in wrist and arms and maximize your tactile sense we have designed XO ODONTO-SURGE with a very light handpiece and hose.

You will not have to look away from the work area to adjust the power. After choosing the operation mode, XO ODONTO-SURGE will do the rest.

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# PRODUCTIVE DENTISTRY

Our objective is to help you increase the value of your equipment investment by saving you time and money, giving you maximum uptime and providing an attractive working environment.

## Always ready to use

No matter if it is small bleeding or full surgery, XO ODONTOSURGE is always ready. With no setup time, only two working modes (CUT / COAG) and automatic power control, XO ODONTOSURGE will quickly become your most efficient tool for everyday treatments.

## Boost your treatments

You can do more during one patient's visit and significantly reduce the treatment time. The dry incisions and coagulation mode are your way to omit the healing period which accompanies the use of a scalpel.

The compact design, ease of use and better treatment results make the XO ODONTOSURGE an absolute must-have for every dental clinic.

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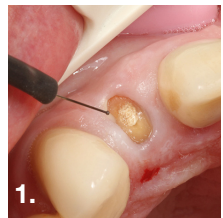
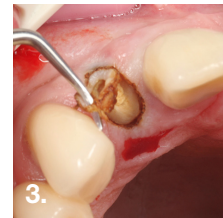
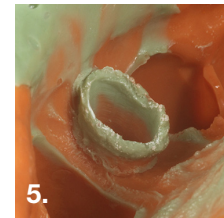
## CASE STUDIES

### EXPOSURE OF CERVICAL ROOT RESORPTION BY DDS. ESBEN KARDEL

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### CROWN EXTENSION BY DDS. ESBEN KARDEL

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# TECHNICAL SPECIFICATIONS

## XO ODONTOSURGE

Stand-alone high frequency electrosurgery unit for soft tissue handling with:

- Maximum output power: 50 W
- Automatic power-tuning system based on the concept of impedance matching
- Work frequency: 27 MHz, yielding no neutral electrode necessary

Operation:

- On/off control in handpiece
- Choice between cut and coagulation via touch buttons on the control unit
- Indication of standby, cut or coagulation by LEDs on the control box

Contraindications:

- Cannot be used near or on patients with implanted cardiac pacemaker
- Do not use near implants<sup>1</sup>

Asepsis:

- Handpiece, hose, handpiece holder, electrodes and electrode casing can be autoclaved at 135°C
- Antibacterial powder coat of the control box

Including:

- Control box with automatic power control
- Power cable
- Handpiece (black)
- Handpiece hose (black)
- Handpiece holder
- Case with standard set of electrodes with both straight and loop-shaped electrodes for facial and intraoral surgery, and ball electrodes for coagulation in casing: 220101, 220102, 220104, 220110, 220113, 220114 and 220121

Dimensions of the control box (W x D x H):

17.5 x 24.8 x 5 cm

Packed in box (W x D x H):

40.5 x 20.5 x 11.5 cm

Weight of box:

3.7 kg

**XO ODONTOSURGE, 230 V (OSU430)**

**XO ODONTOSURGE, 115 V (OSU415)**

<sup>1</sup>XO ODONTOSURGE is a monopolar electrosurgical instrument. As such it is not recommended for surgical procedures in the proximity of metallic structures such as dental implants. Contact between an active XO ODONTOSURGE electrode and an implant can result in a heat-induced injury to peri-implant bone, seriously and permanently damaging the osseointegration and leading to the loss of the implant.

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All rights reserved.
XO® is a registered trademark.
**XO CARE A/S**